



LYSOZYME

DETERMINATION OF LYSOZYME (EC 3.2.1.17) IN SERUM, URINE OR FAECES

Turbidimetric method

Suitable for all analyzers – 200 tests

Product insert with instructions for automated and manual procedures

Faeces Pretreatment Reagent

Lysozyme Standard & Controls



SUMMARY

PRINCIPLE

A suspension of *Micrococcus Lysodeicticus* in a suitable buffer is incubated at 30 °C or 37 °C with the sample. The lysozyme concentration is calculated from the decrease of turbidity. Hen egg white lysozyme is used as a standard.

SAMPLE MATERIAL

Use serum, plasma, urine undiluted or pre-treated faeces.

LINEARITY

Linearity: Plasma / Serum

25 mg / liter at 30°C

13 mg / liter at 37°C

Linearity: Urine

5.0 mg / liter at 37°C

EXPECTED VALUES

In serum: 3.0 - 12.0 mg/l

In plasma: 4.0 - 15.0 mg/l

In urine: 0 - 3 mg/l

In faeces: < 6 mg/l

QUALITY CONTROL

Products

Lysozyme Control Low Level

Lysozyme Control High Level

Product no.

3003

3004

Quantity

10 x 5 ml

10 x 5 ml

QUANTITY OF DETERMINATIONS

According to product insert of Lysozyme Reagent Set (2419):

Procedure

- Manual : 17 tests

- Automated : 200 tests

Products	Product no.	Quantity
Lysozyme Reagent Set	2419	17 manual 200 automated
Lysozyme Standard Concentration: 25 mg/l	2411	10 x 5 ml
Lysozyme Control Low Level	3003	10 x 5 ml
Lysozyme Control High Level	3004	10 x 5 ml
Lysozyme Pretreatment Reagent	2493	1 x 500 ml



NOTES

1. For in vitro diagnostic use only.
2. For professional use only.
3. Always contact INstruChemie for the complete product insert and latest edition.

Printed in the Netherlands, August 2018, Edition 5.0